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INSTRUCTION MANUAL

Read this manual carefully before using the instrument

INDEX

2.	GENERAL RULES.....	PAGE 2
3.	SAFETY.....	PAGE 3
4.	TECHNICAL SPECIFICATION.....	PAGE 3
5.	INDICATOR.....	PAGE 4
6.	INSTALLATION.....	PAGE 5
7.	INSTRUCTION FOR USE	PAGE 6
8.	PRINTER FUNCTIONS.....	PAGE 10
9.	SETTING SETUP.....	PAGE 11
10.	ERROR MESSAGE.....	PAGE 12
11.	MAINTENANCE AND ASSISTANCE.....	PAGE 12
12.	CONFORMITY.....	PAGE 12
13.	IDENTIFICATION LABELS.....	PAGE 13
14.	SCRAPPING AND WASTE DISPOSAL	PAGE 13
15.	WARRANTY.....	PAGE 14

By choosing the **WUNDER** mod. **RW (WU150)** professional electronic scale, you have purchased a high precision instrument.

Since over 40 years **Wunder** has placed its experience at the service of health. This instrument is compliant with national standards in hospitals and clinics with medical class I with measurement function and is calibrated in conformity with accuracy class III.

The instrument is characterised by the possibility of fixing the electronic terminal to the weighing platform by means of a column or else installing the terminal autonomously.

The instrument is equipped with a dual LCD electronic terminal with triple reading to simultaneously view Weight, Height. and BMI

2. GENERAL RULES

**!Carefully read this manual before using the instrument as it supplies
important indications concerning OPERATING SAFETY AND MAINTENANCE.**

The descriptions and illustrations supplied in this manual are not binding.

WUNDER reserves the right to modify its products as deemed convenient in order to approve them without being committed to update this publication.

Conventions:

The following symbols have been used in this manual:

0470 MEDICAL CERTIFICATION

METROLOGICAL CERTIFICATION AND APPROVAL FOR MEDICAL USE

ACCURACY CLASS

ASSISTANCE WARRANTY CERTIFICATE

ATTENTION!
PLACED BEFORE DETERMINING PROCEDURES. COMPLIANCE FAILURE CAN DAMAGE THE PRODUCT

WARNING!
PLACED BEFORE DETERMINING PROCEDURES. COMPLIANCE FAILURE CAN HARM THE OPERATOR OR PATIENT OR DAMAGE THE PRODUCT

WASTE DISPOSAL

TYPE B PARTS SUPPLIED

BATTERY POWER

INDICATION OF WEIGHT FUNCTIONALITY

INDICATION OF STABLE WEIGHT

INTERFERENCE COULD OCCUR NEAR THE APPLIANCES

DUAL INSULATION

3. SAFETY**WARNINGS!**

Operators must read this manual carefully, comply with the instructions it contains and become familiar with the correct use and maintenance procedures of the instrument.

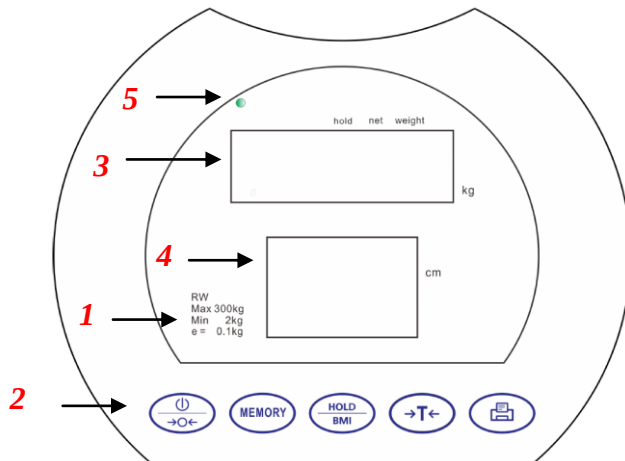
- Retain this manual for consultation and as a help in staff training

- Do not overload the instrument beyond its maximum capacity
- Do not apply loads abruptly.
- Do not press the keys with sharp or pointed objects
- Do not try to open the instrument.
- Do not remove seals from the instrument.
- Do not short-circuit the battery terminals
- Use only the power supply provided by Wunder. Before using it, make sure that the local mains voltage is compatible with the voltage of the adapter shown on the identification plate
- Regularly check the integrity of the instrument's power cord and make sure it does not come in contact with hot appliances
- Make sure that the power cord does not create obstruction hazards
- Unplug the instrument before cleaning it
- Do not place the instrument in water or other liquids
- Perform maintenance and subsequent metric verifications regularly

4. TECHNICAL SPECIFICATION

Model	RW 02
Capacity	300kg x 100g
OIML Approval	Classe III
Weighing Unit	Kg
LCD Display	LCD 20mm con 5 digits + second LDC for Height and BMI
Key Functions	ON/ZERO/OFF, HOLD/BMI, TARA, PRINT, MEMORY
Power Supply	Adapter 12V 2A; Rechargeable battery pack
Operation Temperature	5°C / 35°C

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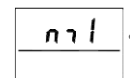
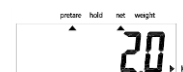
EXAMPLE

8. PRINTING FUNCTION

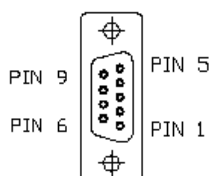
Weight can be printed for records using RS232 interface cable.

After weighing simply press **PRINT** key to print out the results.

The format presented below is the standard format of results print-out and cannot be changed.

Parameters of RS232 interface

Seriale RS232

Connector RS232

WUNDER	
Date/Month/Year	hour:minute
Peso/Weight	30.00kg
Altezza/ Height	0.0cm
Indice Massa Corporea MI(Body Mass Index)	N/A
Sottopeso	
<u>Underweight</u>	<u><18</u>
Normopeso	
<u>Normal weight</u>	<u>18-24.9</u>
Sovrappeso	
<u>Pre-obesity</u>	<u>25-29.9</u>
Obesità calaae I	
<u>Obesity class I</u>	<u>30-34.9</u>
Obesità calaae II	
<u>Obesity class II</u>	<u>35-39.9</u>
Obesità calaae III	
<u>Obesity class III</u>	<u>>40</u>

Connecting with PC

Start Hyper Terminal program from clicking

Start Menu → Programs → Accessories → Communication → Hyper Terminal.

1. New Connection Description

Give new connection a name then click OK.

2. Select Your COM Port

Click Connect to select your COM port. Usually there's only one option for select. Then click OK.

3. Port Settings

Click Bits per second to set up rate at 9600, Data bits at 8, Parity at None, Stop bits at 1 and Flow control at Hardware. Then click OK to complete your setting.

4. Output Data

When press the PRINT key to output data from scale to PC or an

Pin	Signal
2	TX
3	RX
5	GND

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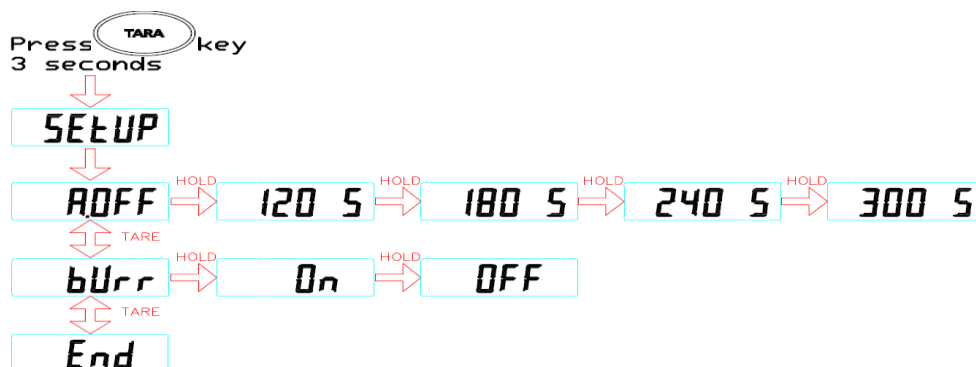
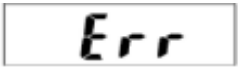
11. MAINTENANCE AND ASSISTANCE

Cleaning the instrument must be done with a soft cloth dampened with water or neutral detergent, avoiding the use of solvents or abrasives.

During transportation, be careful not to subject the instrument to excessive physical shock or mechanical stress. In case of repair or service, contact your dealer

12. CONFORMITY

WEIGHING INSTRUMENT DIGITAL WUNDER: RW(WU150) MODEL SERIAL NUMBER.....

This is to certify that this instrument has been tested and has passed the functional test. Meets the following standards:

**EN 45501 / EN60601-1-2 / EN60601-1
93/42/EEC – Medical Devices Directive**

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14. SCRAPPING AND WASTE DISPOSAL

Disposal  EU 2002/96/EC

This product complies with EU Directive 2002/96/EC. The crossed-bin symbol on the appliance indicates that the product at the end of its useful life, should be disposed of separately from household waste, must be completed in a recycling center for electrical and electronic equipment, or returned to the dealer

at the time of purchase of a replacement. You are responsible for taking the appliance at the end of life at an appropriate collection facility. Appropriate separate collection to start the device resigned to recycling, treatment and environmentally compatible disposal contributes to avoiding possible negative effects on the environment and on the health and promote the recycling of materials of which the product is composed.

For more detailed information covering the collection systems available contact your local waste disposal service or the shop where the product was purchased.

As a consumer you are legally obliged to return used batteries or empty. You can deposit your old batteries at public collection points in your town, or you can place them at any dealer of different kind of batteries that have positioned the bins provided. Even in the case of 'destruction' of electrical and electronic equipment must be removed and stored in special containers.

NOTE: The following symbols are to indicate the presence of harmful substances

Batteries:

Pb Pb = Batteries containing Lead

Cd Cd = Batteries containing Cadmium

Hg Hg = Batteries containing Mercury

Do not dispose of electrical parts and batteries in household waste. Dispose of the battery through collection centers near you.

15. WARRANTY

This certificate must be kept until the warranty has expired.

It must be presented together with the invoice, tax receipt or delivery note providing the name of the dealer and date of purchase whenever a technical intervention is required. Otherwise the user will lose any warranty rights.

The warranty takes effect from the date of purchase and is valid during the entire period foreseen by the current catalogue/pricelist.

By warranty we mean the replacement or repair free of charge of parts making up the appliance which, at the discretion of the manufacturer, are deemed faulty from the origin; Wunder therefore has the faculty of repairing or replacing the item.

The warranty does not cover:

- shipping faults, damage caused by falls, carelessness or tampering
- damage caused by incapability of using the appliance and of its improper use
- damage caused by an insufficient or inadequate electrical system or alterations resulting from environmental, climatic or other types of conditions
- damage due to incorrect installation of the appliance and repairs carried out by unauthorised personnel
- Interventions at home for convenience controls or presumed defects
- Routine maintenance and that which can be considered normal wear from use
- consumables such as: power supplies, batteries, keyboards, plates, wheels, heads, rolls, load cells faulty due to blows or overloads

Service can also be refused when the appliance has been changed or transformed in any way.

In case of interventions at one's home, the customer must pay the fixed fee; if however the appliance is repaired at an authorised Wunder Service Centre, expenses and relative travel risk are borne by the user.

Wunder will not be held liable for damage of any nature caused directly or indirectly to persons, animals or objects resulting from failure to comply with all the instructions indicated in this manual or anyway resulting from improper use.

The Court of Bergamo has jurisdiction in case of any dispute